Instructions for Use:

- Ensure that the patients mouth and airway is clear before usage.
- Remove the product from its packaging.
- Place the transparent face sheet on the face mask.
- Connect the ventilation tube onto the face mask, thus securing the face sheet.
- As taught in First-Aid courses, tilt the patients head slightly backwards.
- With a light pressure place the mask over mouth and nose of the patient.
- The rebreathing process can begin.

Do not forget: Always use protective gloves!

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07-2013

SÖHNGEN® Air-Vita® Bi-Protect

Respiration Mask with Filter and 2-Way Valve

Emergency resuscitation aid for inexperienced helpers in a mask form consisting of different parts to be put together. Air-Vita® Bi-Protect avoids direct contact between the helper and the patient and so helps to overcome inhibitions during artificial respiration. A potential risk of infection can be reduced.

Air-Vita® Bi-Protect consists of 3 separate parts: • the transparent face mask • the transparent face sheet • the ventilation tube with 2-way valve. The transparent mask is very flexible and fits excellently onto any facial contour. The transparent face sheet prohibits any direct contact between the helper and patient. The transparency of both these product components allow the patient to be continually observed during respiration.

The ventilation tube contains two special features • a protective filter that prohibits the transmission of fluids • a two-way breathing valve permitting inhalation and exhalation.

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SÖHNGEN®

PRODUCT INFORMATION

07-2013

SÖHNGEN® Air-Vita® Bi-Protect

Respiration Mask with Filter and 2-Way Valve

Emergency resuscitation aid for inexperienced helpers in a mask form consisting of different parts to be put together. Air-Vita® Bi-Protect avoids direct contact between the helper and the patient and so helps to overcome inhibitions during artificial respiration. A potential risk of infection can be reduced.

Air-Vita® Bi-Protect consists of 3 separate parts: • the transparent face mask • the transparent face sheet • the ventilation tube with 2-way valve. The transparent mask is very flexible and fits excellently onto any facial contour. The transparent face sheet prohibits any direct contact between the helper and patient. The transparency of both these product components allow the patient to be continually observed during respiration.

The ventilation tube contains two special features • a protective filter that prohibits the transmission of fluids • a two-way breathing valve permitting inhalation and exhalation.

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W.Söhngen GmbH Platter Str. 84 Tel.: 06128 873-0